Oslo Bock[®]

QUALITY FOR LIFE

Derma Skin Care Products

Product Information

PERFECT SKIN CARE

The **epidermis** is the barrier between our body and the environment. It is therefore the most important protection the body has against the outside world's harmful effects. To protect against pressure and friction, healthy skin is often thicker in certain areas, such as the palms of the hands and soles of the feet. The **dermis** contains blood vessels and nerves with special receptors that are sensitive, for example, to pressure, touch, pain, hot and cold, making the skin an important sensory organ. The **subcutaneous** tissue is primarily fatty tissue, along with larger blood vessels and thick nerve fibers. The hair roots are also found in this layer along with the sebaceous glands and sweat glands. When the epidermis is intact, the deeper layers of the skin are protected from chemicals and microorganisms.


Epidermis

Dermis

Subcutaneous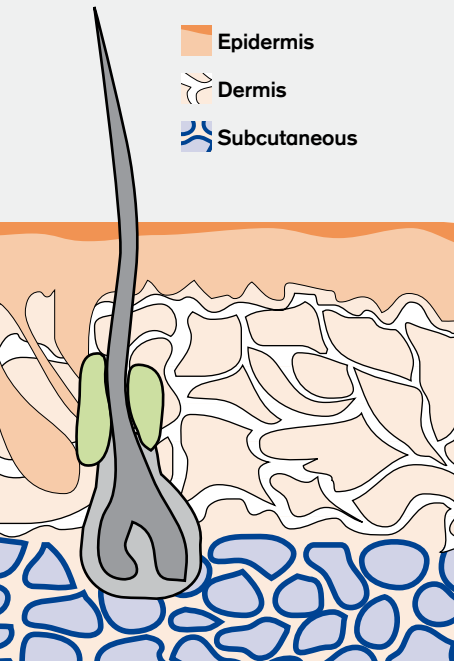

For people wearing prostheses or orthoses, unavoidable mechanical stresses have adverse effects on the skin's function. There is often a close correlation between these causes and effects.

The main problems are:

- Pain with pressure and friction, soreness
- Build-up of perspiration and odour
- Damage to the skin's protective and immune functions
- Excessive hygiene or incorrect cleansing of the weakened skin

What you can do:

The DERMA series offers a selection of skin care products that have been specially adapted to the needs of people who are wearing prostheses or orthoses: **DERMACLEAN**, **DERMAPREVENT** and **DERMAREPAIR**.

DERMACLEAN

DERMACLEAN

- Cleans gently and safely
- Ph-neutral, free of alkali and phosphates
- Anti-bacterial formula

Application: Apply Derma Clean special cleansing lotion to the skin, rinse thoroughly with clear water after washing and dry.

To clean the liner, turn it inside out and wash with Derma Clean. Then rinse thoroughly with clear water and wipe with a clean, dry cloth. Now let the liner air dry.

Ingredients: Aqua, Sodium Myreth Sulfate, Decyl Glucoside, PEG-7 Glyceryl Cocoate, Isopropyl Alcohol, Cocamidopropyl Betaine, Chlorhexidine Digluconate, Citric Acid, Dehydroacetic Acid, Benzoic Acid, Phenoxyethanol, Cetrimonium Bromide, Parfum, Ethylparaben, Methylparaben, Propylparaben, Butylparaben, Isobutylparaben, Limonene.

DERMAPREVENT

- Prevents chafing
- Inhibits contact with external allergens
- Covers the skin with a protective coating
- Leaves the skin soft and supple
- Reduces perspiration and eliminates odour by releasing active substance when needed

Application: Apply Derma Prevent to the skin prior to donning the prosthesis or orthosis. Give a few drops on the palm of your hand and rub into the skin until it is dry. Shake well before using!

Ingredients: Cyclomethicone, Dimethicone, Cyclodextrin, C12-13 Alkyl Lactate, Farnesol, Parfum, Limonene, Linalool, Hexyl Cinnamic Aldehyde, Butylphenyl Methylpropional, Citronellol, Citral, Geraniol, Coumarin.

DERMA REPAIR

- Moisturizes and promotes regeneration of dry, irritated skin
- Reduces the effects of excessive strain and soothes irritated skin
- Antibacterial formula. Helps assist the skin's defense system against harmful environmental effects
- Regulates moisture and makes the skin noticeably more supple and elastic
- Increases skin function, promotes the skin's blood circulation and helps cells to grow

Application: Use Derma Repair special basic skin care on a daily basis and generously apply it to dry, irritated skin after cleaning it with Derma Clean special cleansing lotion.

Ingredients: Aqua, Glycerin, Panthenol, Cetearyl Alcohol, Decyl Oleate, Octyl Stearate, Orbignya Oleifera, PEG-40 Hydrogenated Castor Oil, Tri ticum Vulgare, Hippophae Rhamnoides, Tocophe rol, Lecithin, Tocopheryl Acetate, Allantoin, Glyceryl Stearate, Sodium Hyaluronate, Hydrogenated Palm Glycerides Citrate, Ascorbyl Palmitate, C13-14 Isoparaffi n, Polyacrylamide, Chlorhexidine Di gluconate, Laureth-7, Parfum, Cetrimonium Bro mi de, Phenoxyethanol, Acrylates/C10-30 Alkyl Acrylate Crosspolymer, Butylparaben, Ethylparaben, Isobutylparaben, Methylparaben, Propylparaben, Sodium Hydroxide, Limonene, Linalool, Hexyl Cinnamic Aldehyde.

All Derma Skin Care Products are only intended for external use. Avoid any contact with the eyes and do not use in case of open wounds or infections. Dermatologist tested.

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